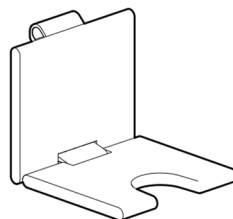
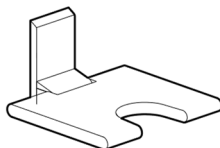
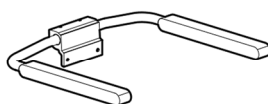
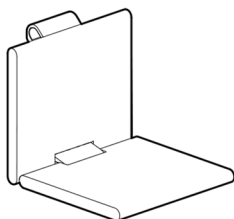
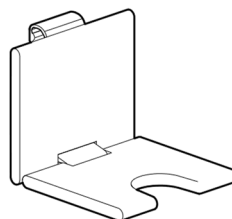
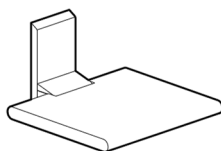
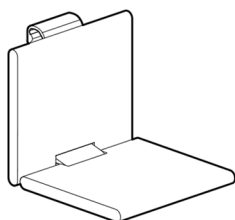
NORMBAU

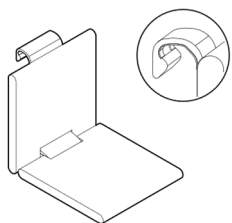
PASSION FOR CARE

Cavere[®] Care
Inox Care[®]
Nylon Care


- DE** Montage- und
Gebrauchsanleitung
EN Fixing instruction and
directions for use
FR Instructions de montage et
d'utilisation
NL Montage- en
gebruiksaanwijzing


Weitere Informationen
Further information
Plus d'informations
Meer informatie
extranet.normbau.de


7847 380
7847 381

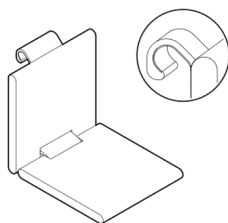
0847 710
0847 910

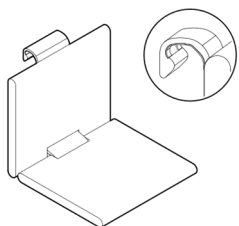
7847 450
7847 451

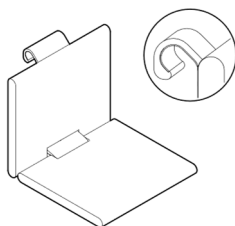
0847 714
0847 914

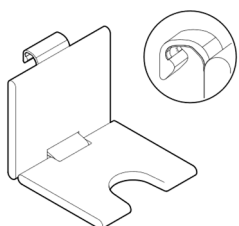
7847 455
7847 456

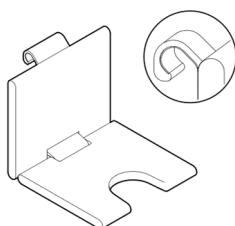
0847 719
0847 919

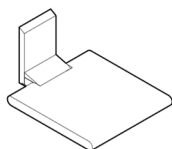
7844 381
7844 382

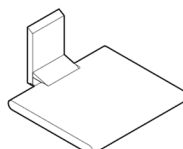7844 451
7844 452

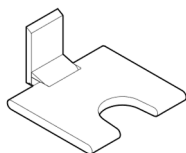7844 456
7844 457

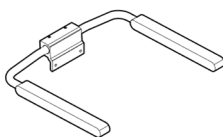7843 100

Vario System
Vario system
Vario système
Vario systeem

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Intended use

NORMBAU products are designed for use in wet rooms, at home, in care facilities and hospitals. They offer support to people with and without physical limitations during showering.

Disclaimer of liability

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions.

Warranty

The statutory warranty period is 2 years.

NORMBAU provides a total of 5 years warranty on functionality and stability.

If the intended operating conditions and the maintenance and cleaning guidelines are respected, the expected service life is at least 10 years. The product has been tested and validated based on this service life.

Warning notices

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009.

Risk of injury

The product surfaces can become very hot when exposed to external heat sources, e.g. direct sunlight or contact with hot water.

Risk of injury

If the max permissible user weight is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

Risk of injury

When turning up the suspendable shower seat, take care that your fingers are not squashed between seat cushion and back cushion.

Safety instructions

Installation is to be carried out by a specialist and only with NORMBAU mounting sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max. loading permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at www.normbau.de.

Load

Max. loading permitted of listed products: 150kg

Maintenance

NORMBAU products are not maintenance-free! The products are to be tested monthly by qualified specialists for proper function. In the event of defects (e.g. insufficient braking force), the assembly instructions must be observed.

Care and cleaning instructions

⚠ Risk of injury

Improper cleaning can cause damage to the surface and lead to injuries.

For hygiene reasons and to ensure proper cleaning in accordance with the instructions, hanging seats must be removed from the shower handrail regularly to maintain a healthy surface structure and a durable coating.

Cavere Care

Powder-coated products from NORMBAU have a smooth, easy-to-clean surface. In order to maintain the decorative design and color of the coating of our products for years, cleaning and care must be carried out professionally. Slightly adhering dirt can be removed with a sponge and water to which a neutral cleaning agent (e.g. dishwashing liquid) is added. Immediately after each cleaning process, rinse with pure, cold water. A chamois or microfiber cloth removes the remaining wiping marks or drops of water. Abrasive or strong acidic/alkaline cleaning agents should never be used. With metallic coatings, heavy soiling should be cleaned as soon as possible, since dried and old dirt can often only be removed abrasively and with the risk of damage to the powder coating or a change in color. Common wiping disinfectants can be used.

PUR-integral foam

The upholstered seats should ideally be treated with mild, over-the-counter cleaning agents and disinfectants with low alcohol content. Agents containing iodine can lead to staining, and long exposure times for coloured care products should always be avoided. Avoid exposing upholstered seats to temperatures over 80°C.

More information about
maintenance and cleaning

Disposal

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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GB

Pre-wall installations with panels and integrated mounting plate made of hardwood.

Lightweight walls, made of gypsum, flake board and hardboard with a backing made of at least 30 mm thick multi-layer glued wood.

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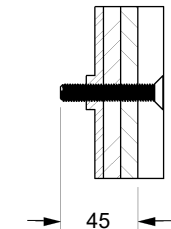
GB

Pre-wall installations with steel mounting plate.

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

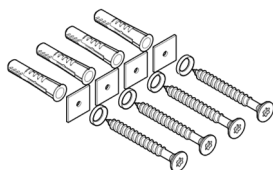
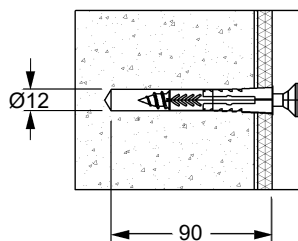
GB Masonry made of solid stone:
concrete \geq B15, solid brick \geq Mz 12,
sand-lime brick \geq KS 12,
natural stone with compact grain

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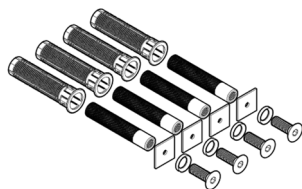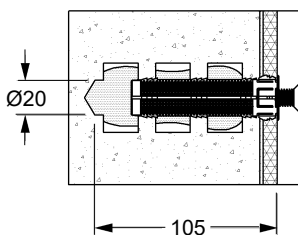GB Masonry made of perforated stone:
perforated brick HLz \geq 12, cavity brick KSL \geq 12,
checker brick \geq Hbl4, Hbn4, other hollow brick
materials, concrete / lightweight concrete,
solid brick \geq Mz 12, sand-lime brick \geq KS 12,
natural stone with compact grain, solid pumice

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